







DON'T FORGET...

- Aseptic technique and proper skin preparation of the site are essential.
- Discard BD SafetyGlide[™] Syringe in the nearest sharps collector. Follow your institution's policy for safe disposal of all medical waste.

BD SafetyGlide™ Allergy Syringe Tray



1 mL Allergy Tray with Permanently Attached Needle For Treatment

| I | CATALOG# | NEEDLE | BEVEL | PACKAGING |
|---|----------|----------|------------------|----------------------|
| | 305950 | 27G 1/2" | Regular Bevel | 25/Tray - 1,000/Case |



1 mL Allergy Tray with Permanently Attached Needle For Testing


| CATALOG# | NEEDLE | BEVEL | PACKAGING |
|----------|----------|----------------------|----------------------|
| 305951 | 26G 3/8" | Intradermal Bevel | 25/Tray - 1,000/Case |

1.888.237.2762



BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. ©2003. BD 7/03-0482

BD Medical Systems 1 Becton Drive Franklin Lakes, NJ 07417 www.bd.com

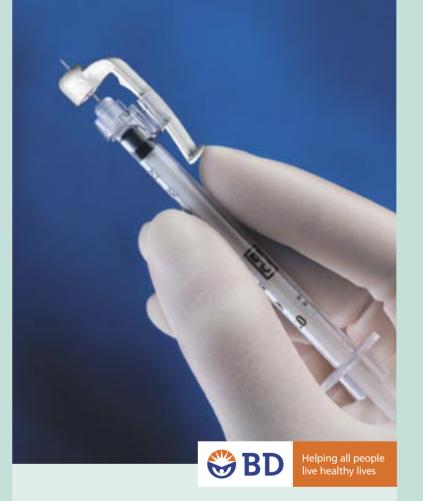

Safety in Small Doses

BD SafetyGlide[™] Allergy Syringe Tray



Aspiration

- Aspirate medication into syringe as per your usual technique.
- Rotate safety arm for scale readability.

Injection

- Administer injection following established technique.
- To facilitate a low-angle injection, safety arm can be rotated so it is oriented to the needle bevel.

Activation

 After injection, immediately apply a single finger stroke to activate the safety arm by moving it completely forward.

EASE OF USE

- Single-handed activation
- Minimal change in technique
- Rotation of safety arm allows for low angle injections

SAFETY

- Secure and visible lock
- Finger stays behind the needle at all times

PATIENT CARE

 Activation occurs after needle is withdrawn: no negative impact to the patient

EFFICIENCY

- Permanently attached needle means minimal dead space – minimal drug waste
- Available in both Testing (Intradermal bevel) and Treatment (Regular bevel)

BD SafetyGlide™ Allergy Syringe Tray Safe. Small. Simple.

PRIOR TO ACTIVATION



AFTER ACTIVATION



The evolution of safetyengineered injection.

BD, the industry leader in safety-engineered injection, offers the most extensive array of safety-engineered injection products available. With a record of innovation spanning more than a century,



BD's continued investment in safety technology shows our commitment to providing innovative products and solutions for the future.

Each case of BD SafetyGlide Allergy Syringe Trays includes:

Purple BD Allergist Advantage Coupons
that are worth <u>double</u> than the
green conventional product coupons.

A postcard for a FREE BD Allergy Syringe Organizer



It's back by popular demand — this organizer conveniently arranges the syringes by number so they are ready for injection.

New Improved Cap – easier to grip and pull off; easier to recap for simple transport.